



IntelliGaze was tested with the Antivirus Tool Windows Defender. Other tools such as AVG, Antivir, McAfee or Kaspersky can cause unpredictable behavior. Use them without support at your own risk.

**IntelliGaze v11 requires Windows 11 or Windows 10.
IntelliGaze will run in trial mode on older cameras without license upgrade. Please check your license status in advance.**

Version 11.0.8661 – (18.9.2023)

New

- QuickStart guide updates

Fixes

- Com Interface of the API reworked to improve forward compatibility

Version 11.0.8636 – (24.8.2023)

Fixes

- improved camera reconnect behaviour

Version 11.0.8595 – (14.7.2023)

New

- tracker improvements for Cam30NXT devices

Version 11.0.8563 – (12.6.2023)

Fixes

- license of Cam30NXT could not be updated
- Gaze Assistant crash when exporting XPS report
- keyboard shortcuts not localized
- closing OSK with the X button resets the keyboard modifier

Version 11.0.8528 – (8.5.2023)

New

- close button on OSK
- sentence history in OSK can now be cleared with ResetIntelliGaze
- keyboard layout selection is now persistent
- infobar message when a calibration is loaded or saved
- DisableModernStandby.reg added to /tools to fix the wake up from standby issue in Microsoft Surface tablets (optional)

Fixes

- camera reconnect issues fixed
- scaling issues on smaller monitors
- keyboard font size changes
- offscreen activations are now inactive in live mouse mode when paused
- service tool crash, privacy policy
- dwell feedback off turned off clicks
- disable keyboard shortcuts no longer requires a program restart
- crash after monocular calibration

Version 11.0.8472 - Beta (13.03.2023)

New

- gaze tracking performance improvements

- cpu load reduction
- new API call to stream more eye parameters to client applications
- debouncing dwell buttons
- prevent repeated click option for dwell buttons
- larger magnifier
- gaze tracking performance improvements
- new command line parameter "nosplash" to prevent splash screen

Fixes

- cam nxt didn't show the correct total up time
- reconnect issues with all cameras on some platforms
- autostart wasn't applied before the next intelligaze start
- cpu load reduced when showing live camera view
- tracking status not properly initialized after program start
- fix for ThirdParty application launching IntelliGaze with "background" parameter

Version 11.0.8419 - Pre (19.01.2023)

New

- gaze buttons now have a click animation
- header in all option dialogs
- can now lock options for users
- launch calibration from options
- calibration star rating
- pause visualization
- disable offscreen overlays when system is paused
- auto stop when doing eye and desktop recording
- option to debounce clicks on gaze buttons
- larger offscreen region
- gaze buttons better accessible when offscreen region is right next to the screen

Fixes

- no audio feedback when doing zoom click DnD
- IntelliGaze clicks without valid data
- selecting task in taskmanager doesn't close home
- zoom clicks in corners improved
- larger zoom region
- high gpu load when camera view not visible
- camera view was top most and covering pop up dialogs
- keyboard offscreen overlay not visible when keyboard is open
- desktop is not cleaned (side bar) when API client connects to IntelliGaze

Version 11.0.8271 - Pre (13.10.2022)

New

- show numbers as targets in gaze verification
- calibration process is now touch friendly
- zoom click visualized
- 0 point calibration

Fixes

- scroll overlay now DPI aware
- Keyboard shortcuts handling in app profiles (key recording) ignored
- high processing load of IG11 and Cam30NXT
- side menu was sometimes visible in home screen
- options and home is closed now when camera was disconnected
- several GUI scaling issues fixed on tablet
- skip bad points during calibration fixed
- search camera dialog can't be closed with touch
- improve and retry in calibration result dialog fixed
- memory leak when using precise clicker
- click noise on gaze over buttons removed

Version 11.0.8271 - Pre (24.08.2022)

New

- confirmation question when exiting IntelliGaze
- mouse action switched back to left after right, double left or dnd
- mouse actions are now lockable
- show volume in percent on home screen
- call help function
- click animation
- windows button on home screen
- optional help text on op bar
- CPU and GPU performance optimized
- flat front option in set up wizard

Fixes

- layout of calibration result dialog is messed up on Gridpad
- searching camera message is too sensitive
- voice selection is not persistent
- keyboard opens although no data is available
- GUI layout issues fixed
- keyboard doesn't open in rare cases

Version 11.0.8223 - Pre (07.07.2022)

Note: This version drops Windows 8 and Windows 7 support.

New

- Cam30NXT support
- new desktop control

Version 5.5.8382 - (13.12.2022)

New

- support for new camera Cam30NT v1.5
- support new camera driver 4.96

Fixes

- logging USB low voltages on SB1x platforms

Version 5.5.8174 - (19.5.2022)

New

- support for new camera driver 4.95 fixing the problem that waking up from sleep can create a BSOD occasionally
- support for Cam30NT and SB12OEM with serial numbers: IC3x-n.nn-X-nnnn-aaaa

(refer to Product Change Note 220411_A)

aaaa ≥ '2218'

Version 5.5.7881 - (30.07.2021)

New

- API – data support for Tolt Technologies ability-drive

Fixes

- in rare cases in-operational Operation Center pops up when Grid is launched
- crash when Windows has no audio device available

Version 5.5.7761 - (1.04.2021)

New

- new app, Telegram, PDF Viewer, Youtube Music (experimental)
- improved browser app (autoscroll function, integrated PDF View, easy accessible search option)
- Profile support for Look to Learn 2021

Fixes

- Click with keyboard switch (Neuronode), didn't work with magnifier
- Mouse click events not working when browser is zoomed
- Launcher page not gaze accessible after edit

Version 5.5.7716 - (15.02.2021)

New

- Russian keyboard layout and word prediction
- Profile support for Look to Learn 2021

Fixes

- shift key on OSK is clicked but keys are lower case after page flip

Version 5.5.7635 - (26.11.2020)

New

- ICC now supports apps (Browser, Swifttalker) and cloud apps (WhatsApp, Facebook, Gmail, Youtube, Twitter)
- **Existing ICC content will be incompatible with this version**
- OSK with emoji, sentence history
- new API to stream eyeball position and IntelliGaze status
- Skip bad points calibration option now uses a timed calibration (calibration feels smoother but can fail if the user is not attentive)
- Default cam standby time set to 5min

Fixes

- Status windows pops up after calibration although hidden by API
- Status window eye camera view now shown is correct proportions
- IntelliGaze quits when pressing the "@" on virtual keyboard
- Incorrect voice settings fixed

Version 5.1.7475 - (19.06.2020)

Fixes

- Bad accuracy under head movement on Gridpad12 and 15
- User Interface scaling didn't work on Gridpad12 and 15
- Bluetooth keyboards can now trigger clicks
- Changing keyboard trigger doesn't require IntelliGaze restart anymore

Version 5.1.7346 - (11.02.2020)

New

- Support for the Gridpad12 device
- improved calibration with partially covered eyes
- animated calibration now API accessible
- improved eye video display
- activate camera view in status window of Grid with touch
- OnScreenCommunicator profile
- Support for 64bit native C API apps (Python, Matlab, etc.)
- Support for GEWA environmental control
- Cam driver update to 4.92 to increase compatibility with Windows 10

Fixes

- Audio synchronization issue with desktop recording
- Shrinking rectangle gaze cursor fixed
- Side menu doesn't disappear when pause is activated from operation center

- Modern UI apps now in IntelliGaze task manager visible
- Calibration timeout triggered too early when doing a manual or Grid3 calibration
- Calibration timeout triggered too early when doing a manual calibration from within Grid

Version 5.0.6946 - (7.1.2019)

Fixes

- Task manager no longer shows full screen start menu button
- Installer fails to install when a newer VC redistributable was installed
- Several scaling issues in preferences at 150 DPI
- Scrollbars now touch friendly
- Position helper D&D and magnifier not working when switching between modern UI and standard app

Version 5.0.6873 - (26.10.2018)

Fixes

- very seldom IntelliGaze doesn't start with the message that the license must be renewed although license didn't expire
- Esc on physical keyboard loads the default profile
- possible crash when using the alea keyboard and quick communication
- crash when resizing Chrome Browser or WhatsApp desktop app
- WhatsApp not scrolling contact list
- long touch input can close ICC
- Calibration window is behind Rolltalk UI

Version 5.0.6831 - (14.09.2018)

Fixes

- IntelliGaze task list shows IntelliGaze
- Voice settings not saved
- Audio feedback not working when pinning modern UI side menus
- Pause button resets pinned menu
- Side menus disappear faster when not looking at them
- Sound bug in operation center after the alarm sound was played once
- Position helper and magnifier doesn't work when mouse mode is set to mouse emulation or fast mouse
- Position helper and reading scroll doesn't work in modern UI apps
- System gesture button doesn't work in modern UI apps
- Touch gestures don't work in modern UI apps
- Edit cell is sluggish in some combinations of display DPI - display resolution

Version 5.0.6813 - (27.08.2018)

Fixes

- shrinking rectangle gaze cursor was invisible when blink activation was chosen
- some Monitor Calibration values which worked in previous versions of IntelliGaze didn't work anymore
- desktop recording crashed when Window10 blocked access to the mic hardware
- IntelliGaze blink detection parameters are not reset when an API application disconnects

Version 5.0.6780 - (25.07.2018)

Fixes

- improved stability of Gaze Assistant
- animated calibration target wasn't working in Assisted Mode in some cases
- API calibration (Grid 2 / Grid 3 Manufacturer Calibration / Rolltalk) no longer uses animated calibration target by default
- calibration window is always on top again

Version 5.0.6775 - (20.07.2018)

New

- improved calibration, smoother movement of target
- live preview of calibration target in preferences dialogue
- new animated calibration for children

- localized dialogs are now fully DPI aware, lengthy translations are no longer cut off
- show hint in profile pop-up windows if no secondary trigger is enabled but supported by profile

Fixes

- cursor artifacts when moving cursor slightly above the screen boundary in Windows 8.1.
- unstable USB connection is now reported as such, formerly a missing license was reported
- an auto-update deletes ICC pages in some cases

Version 5.0.6660 - (27.03.2018)

Fixes

- Installer didn't uninstall all files
- Installer repair now actually repairs a broken installation
- Update from IntelliGaze between 5.0.6618 and 5.0.6645 to any newer version breaks the connection between Grid3 and IntelliGaze, restart the installer and click "repair" to fix the problem

Version 5.0.6645 - (12.03.2018)

Fixes

- the cursor and status windows behavior setting could not be changed in assisted mode
- operation center didn't open with touch in some application (Grid3 with mouse input)

Version 5.0.6628 - (23.02.2018)

Fixes

- IntelliGaze crashes when a Grid calibration was started and canceled in Grid3 on a PC which never did an IntelliGaze Calibration
- Download button not visible on the ReleaseNotes of version 5, download this version from the IntelliGaze website if you experience this problem
- Online help not shown in error feedback form
- Error feedback windows immediately disappeared
- ICC: drag n drop of tracking status cell doesn't work
- ICC: tracking status cell cannot be selected for editing in gaze editing mode

Version 5.0.6618 - (13.02.2018)

New

- can now queue IR commands to select 2 or 3 digits TV channels with a numeric keypad

Fixes

- ICC: preview of environmental control commands didn't work in cell editor
- ICC: could not restore layout when a read only image was backup
- ICC: could not use ResetIntelliGaze when a read only image was used in a layout
- ICC: icons on toolbar were too small on HiDPI displays
- ICC: creation of buttons on previously deleted and recreated pages didn't work
- ICC: Environmental control can be used simultaneously by Grid and IntelliGaze
- can now use version 5 on Windows 8.1
- reduced number of reboots when installing / updating the driver

Version 5.0.6590 - (16.01.2018)

New

- integrated multilingual, eye editable communication pages with integrated icons, optional symbols and environmental control
- redesigned side menus to unify user experience in modern UI and desktop
- side menus stay open for a moment to ease selection
- bright keyboard schema available
- robust dwell clicking on gaze buttons and keyboard
- external tool to edit / import / export the user dictionary of the word prediction
- shift key can toggle capitalization of predictions
- prediction resets when the input focus is set to a different context
- keyboard offers capitalization on predicted words

- general improvements of the keyboard and word predictions
- Greek keyboard and word predictions
- idle enable tool added to increase battery lifetime on tablet computers
- monitor Off API call
- extended Calibration Result API call to display binocular calibration results in client apps
- redesigned warning messages, system is not "over-warning" anymore to not disturb the user
- audio device can be selected for speech output
- system gestures and offscreen swipes are now disabled in desktop mode of Windows 10
- FPS counter in camera view to analyze timing issues on a PC
- Dutch translation

Fixes

- pause symbol in status windows was active although an API application was in control of the cursor
- default keyboard layout was active also it was unchecked in the preferences
- camera did not connect when the plane was not measured
- crash when using the magnifier, very seldom IntelliGaze would crash if the magnifier was opened
- IntelliGaze fails to start integrated Windows OSK on 64bit
- collect logs tool wasn't working
- auditive click feedback wasn't working in Windows 10 apps
- switch to cursor shapes (default profile) didn't work
- start menu scrolling in Windows 10 didn't work anymore
- rectangle dwell feedback didn't work properly
- task manager didn't work when the Windows OSK was used

Version 4.2.6389 - (29.06.2017)

New

- improved driver compatibility with Windows 10 anniversary update

Fixes

- open operation center with touch didn't work when IntelliGaze was put into Windows autostart with the tool "CreateAutostart" or with autostart option in the application center
- fixating objects in the Windows environment was difficult on retina displays (hi res displays)
- DPI issue with magnifier on retina displays (hi res displays)
- monitor calibration tool with option to measure "flat front" set-ups

Version 4.2.6122 - (5.10.2016)

Fixes

- sleep mode not working after Windows10 anniversary update (home edition)
- after a failed calibration a misleading calibration result was shown
- loading a calibration altered the calibration preferences
- IntelliGaze lost calibration after changing some calibration preferences
- API Clients don't connect after Demo Mode
- API Clients don't connect after sleep or hibernation
- crash when eye image view was open for too long
- possible crash when side menus were opened with keys in demo mode
- false hardware error 3 after disconnecting and connecting the camera again

Version 4.2.6024 - (29.06.2016)

New

- Support for Onyx PC touch display
- send a single key as trigger action to use scanning in AAC software
- option to prevent double activations in trigger regions

Fixes

- Popup windows on keyboard trigger too fast
- text preview window at wrong position in high DPI modes
- Calibration settings were lost when switching between Grid3 and IntelliGaze assisted access
- Windows OSK did not work in 64bit system

Version 4.2.6008 - (13.06.2016)

New

- OSK extensions: word prediction, word correction, learning dictionary
- new Polish keyboard layout
- custom keyboard click sounds
- OSK in 3 different sizes available
- OSK can be kept open with a lock function
- Communication Page: quick speech generation using the SAPI5 voices (Windows 10 or custom)
- Tellus 5 and Motion Computing devices can use hardware buttons to start a calibration
- pause symbol in status window
- Gaze Assistant report improved
- options dialogs are now DPI aware
- scalable Operation Center
- online update improved
- improved wake up from user sleep

Fixes

- cursor shape setting was not persistent in Assisted Mode
- option to always hide the status window in Assisted Mode
- performing a calibration in Grid2/Grid3 reset the calibration settings in IntelliGaze

Version 4.1.5814 - (02.12.2015)

New

- Windows 10 support, its recommended to use Windows 10 in tablet mode
- separate tool to verify the accuracy
- downgrade license now creates a termination code
- demo mode to emulate gaze data with mouse (Ctrl+F12)
- help functions to open side menus and keyboard with touch
- support for direct mouse click hardware to replace dwell clicks

Fixes

- "cable too long " message was falsely triggered
- power supply errors are now reported properly
- some setting where lost after an update

Version 4.0.5711 - (21.8.2015)

Fixes

- Italian labels where not in the right place on some dialogs
- Logging messages added and fixed
- Pause was not always reliable started when a physical mouse was moved
- Windows cursor is now enforced when using "smart cursor" and disabled in "raw" or "mouse emulation" mode
- Could not assign a new action to the primary trigger
- Invisible useless blickfang was left on the desktop after opening the operation center options

Version 4.0.5661 - (2.7.2015)

New

- Cue cursor (blue dot) can be made large
- Pause animation unified in operation center and side menu
- Option to use a large font on OSK
- Operation Center can be shows in application center (to pause in safe mode)
- Can now accept a calibration with keyboard (keys 1,2,3 to accept, improve, retry)
- Fall back to alea OSK when third party keyboard is not available
- Switch preview in input selection of the preferences
- Profile editor now with 2 extra buttons to navigate faster through profile list
- Elements in profile editor are now easier to hit with gaze

Fixes

- App can't be renamed in application center
- White background color of some modern UI apps hide foreground text
- Click noise not working when clicking on "current application" tab
- Scroll function not working in mouse emulation mode
- Wrong status window size in Grid2 on high DPI settings
- Alea OSK keyboard layout selection is now persistent

Version 4.0.5641 - (12.6.2015)

New

- Clicking X in the calibration accept dialog restores previous calibration, countdown timer to restore previous calibration in calibration accept dialog when gaze activated
- Gaze controls menus (offscreen menus, operation center) are now scaled to a fixed physical size independent of the windows DPI setting, optional enlargement of gaze controls in service mode
- Calibration will exit automatically after 30secs of inactivity, previous calibration will be restored
- Actual dwell time in ms now shown in IntelliGaze options
- Dasher profile
- Multitext was not shown in the task manager of the operation center
- Missing translations in confirm quit dialog
- Calibration accept dialog can't be closed with eye anymore if calibration was started with touch
- License upgrade from an old "standard" license to any new IntelliGaze v4 license at the start of Intelligaze didn't work
- Deactivation of keyboards (integrated, win7, third party) didn't work

Version 4.0.5563 - (26.3.2015)

Fixes

- Windows cursor disappeared when closing Grid2 and IntelliGaze was started by Grid
- Background color of some icons in application center was white
- Modern UI app did not close properly on Windows 8.1
- Accuracy offset after opening the Windows 8 integrated keyboard in the magnifier
- Operation Center volume setting was not saved when "maximize call help volume" was active
- Trigger activation by blink in gaze regions was unreliable
- Skype, automatic keyboard in chat window
- Internet Explorer, Firefox keyboard opens on edit fields only
- Operation Center opens only when gazing in the middle third above the screen
- larger Blickfang areas in confirm quit and user sleep dialogs

Version 4.0.5533 - Release Candidate (24.2.2015)

New

- Fast cursor (previously Raw Cursor) now snaps into Blickfangs
- Pause in Operation Center of Safe Access
- Eyevideo view is now mirrored

Fixes

- Wake up from sleep no longer freezes camera on some computers
- Gaze cursor was visible at start although no eye was tracked
- Couldn't toggle status windows to eyevideo when tracking not active
- Prevent dwell animation outside operation center when used with gaze

Version 4.0.5518 - Beta (9.2.2015)

New

- new appearance option to configure status windows behavior
- automatically close operation center after unpause
- access IntelliGaze preferences from facilitated mode
- optionally maximize call help volume
- user calibration marked in operation center of calibration changed
- gaze initiated calibration superseded by operation center start calibration
- keyboard shortcuts to operate facilitated operation center
- monitor calibration autodetects screen size
- support for DPI up to 150%

- Export profiles tool
- Keyboard trigger now accepts more keys to work with switch buttons which can't emulate F13-F24 keys

Fixes

- Operation Center could not be opened with touch on recent tablet pcs such as Panasonic, Motion Computing, Sahara Paceblade, Tellus4
- status was not show at the right position in Grid2 when DPI was no set to 100%
- status disappeared in Grid2
- external screen wake up also wakes up IntelliGaze
- camera connection error after wake from standby
- couldn't toggle eyevideo when tracking not active

Copyright © 2007-2023, alea technologies gmbh